





# ONE TECHNOLOGY ENDLESS APPLICATIONS


# THE JETPEEL INNOVATION

JetPeel is a revolutionary, non-invasive aesthetic treatment system offering a needle-free, pain-free solution for transdermal infusion. It combines lymphatic drainage, exfoliation and infusion to treat a variety of skin conditions on the face, body and scalp.

Powered by jet aviation engineering, JetPeel touch-free patented handpieces use pressurized air, accelerated to supersonic velocities, to open microchannels in the skin to deliver powerful, bioactive skin nutrients deep down into the dermal layer. Proprietary Jet Solution serums containing anti-aging plant stem cells, antioxidants, hyaluronic acid and biomimetic peptides offer a broad range of personalized, results-based treatments.

# THE JETPEEL ADVANTAGE

In comparison to competitor handpieces, JetPeel nozzles penetrate significantly deeper into the skin.

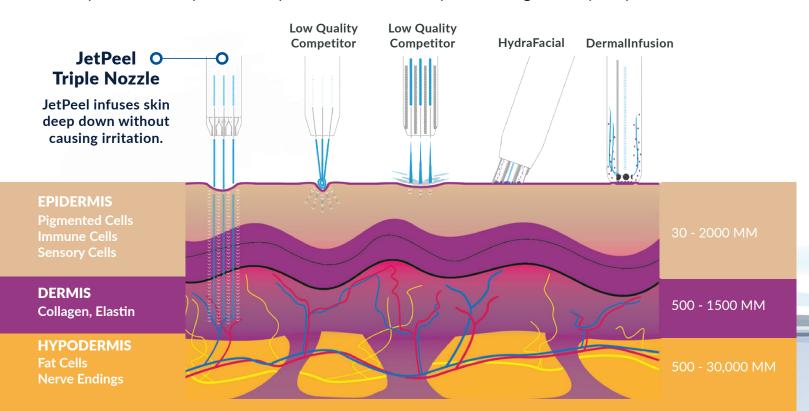

**Advanced Facial Treatments Chemical Exfoliation | Microdermabrasion** Mesotherapy | PRP Therapy Lymphatic Drainage & Tissue Massage **Excessive Sweating | Hair Restoration** 

# THE JETPEEL **BUSINESS OPPORTUNITY**

### ONE VERSATILE TECHNOLOGY

Allows practitioners to administer a wide range of treatments for face, body and scalp.

### NO NEEDLES, NO PAIN, NO DOWNTIME

Offers a holistic, natural approach to microneedling, injections, and cosmetic procedures.

### **EXCEPTIONAL ROI**

Results-based treatments, with low consumable cost and premium price point, yield a rapid return on investment.

### SAFE AND EFFECTIVE

Class 1 FDA registered device, available in both medical and aesthetic versions, offers customizable treatment programs for all skin types & skin conditions.

### **ENHANCES RESULTS OF OTHER PROCEDURES**

Can be combined pre and post procedure to improve healing and results of ablative and non-ablative lasers, cosmetic surgery and skin tightening treatments.




# **LOVE JETPEEL**

"JetPeel is needle-less microneedling - way more than a facial." Dr. Dennis Gross, Dermatologist & Founder Dr. Dennis Gross Skincare

"Versatile and effective! People love the results." Director, The Spa by Equinox Hotels

## THE SCIENCE BEHIND THE TECHNOLOGY

A unique patented technology, based on jet aviation engineering, JetPeel uses pressurized oxygen to accelerate liquid or saline to a velocity reaching 200m/sec, and with a special patented handpiece nozzle, breaks down the accelerated liquid into micro-droplets. The powerful jet-stream of micro-droplets stretches the skin at the point of contact causing barophoresis - creating tiny microchannels in the epidermis to allow for delivery of nutrients down to the dermal layer. This process of dermal infusion, also known as hydroporation, provides a versatile and precise means for performing a wide variety of skin treatments.

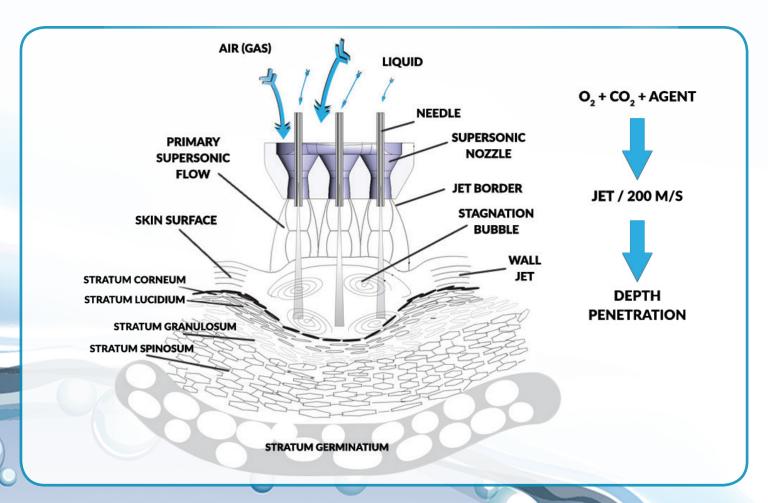


# **THE CLINICAL RESULTS**


In vitro epidermal and dermal histological characteristics in response to hydroporation using JetPeel dermal infusion.



Healthy epidermis before JetPeel infusion.

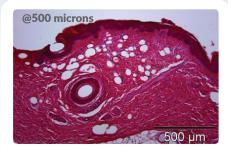

Hydroporation over 30 sec in a defined area led to vacuole formation, removal of the stratum corneum and circumscribed epidermal loss. Localized separation of dermal fiber bundles are visible.

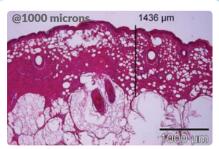

Infusion of JetPeel bio-nutrients led to a uniform vacuole formation, spreading of dermal fibers within the full depth of dermis up to 1436 microns of depth was measurable.

# THE JET ROUTINE THREE-STEP PROCESS

A complete JetPeel treatment takes approximately 45 minutes (depending on protocol) and includes lymphatic drainage, exfoliation, and a customized infusion of natural, bioactive skin nutrients to address specific skin conditions, providing visible results with no downtime or skin irritation.



### **Lymphatic Drainage**

A pressurized massage using specially-formulated detoxifying water containing Aloe, Silver Oxide and Hyaluronic Acid, deep cleans and purifies. This stimulates blood and lymphatic circulation to eliminate toxins.



### **Exfoliation**

Detoxifying water, combined with active exfoliating ingredients such as Glycolic and Mandelic Acid, gently removes natural skin debris and dead skin cells. It oxygenates tissue and stimulates microcirculation, preparing skin to accept nutrients during the infusion step.



### Infusion

A pressurized force of customized, bioactive skin nutrients\* are infused deep into the dermis via a patented handpiece. All-natural, nutrient-rich formulas include Hyaluronic Acid, Biomimetic Peptides, Vitamin A+E, Vitamin B5, and Vitamin C.

\*Skin nutrients are customized to specific skin conditions.



## **Booster (Optional Add-On)**

An optional infusion step, using a selection of targeted boosters, to address specific skin conditions and enhance treatment results.

# WHERE IN THE WORLD

Launched in 2004.

JetPeel devices are currently
distributed in over 23
countries worldwide.

### **ORIGINS OF JETPEEL**

Manufactured in Israel by TavTech, the JetPeel technology originates from the principles of jet aviation and was developed by a Russian rocket scientist. It was initially used in hospitals for wound cleansing and debridement to treat wound and burn victims. Using patented handpieces, and proprietary Jet Solution serums, the device was later adopted for use in aesthetics.

### **POWER SERUMS**

Jet Solution proprietary serums are manufactured in Milan, Italy under strict European regulatory guidelines.

# **PATENTED HANDPIECES** FOR A WIDE RANGE OF **APPLICATIONS**

JetPeel nozzles penetrate significantly deep into the skin, down to the dermal level. **Exclusive JetPeel patented** handpieces are precisely adjusted to the unique transport mechanism.

The nozzles are able to withstand enormous pressure, in combination with a speed of up to 200m/s. The perfect interaction between the JetPeel handpieces, and the molecular weight of the Jet Solution serums, ensures optimal results.






### **SINGLE**

**TRIPLE** 

**MAGIC** 

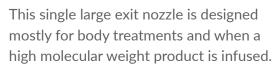

With three specially designed nozzles, this

handpiece is one of the secrets behind the

efficacy of JetPeel technology. Ideal for performing lymphatic drainage, exfoliation

and infusion steps. The perfect tool for

A unique single nozzle, designed

such as fine lines and wrinkles.

for infusion of active ingredients to

effectively treat precise, targeted areas

most face, body and scalp jet treatments.



### **VACUUM**

A vacuum nozzle, designed for removing comedones and other debris from the epidermis, as well as for under eye drainage massage.



### **PRESTIGE**

A special handpiece designed for comfortable infusion of 4 ml single use vials.



### **PRESTIGE PLUS**

Sterile, single patient use, patented handpiece includes an air shield for safe and effective delivery of liquids in a luer-lock syringe without touching the skin. Indicated for use with PRP therapy by authorized practitioners. A no pain, no downtime alternative to microneedling and injections.





Fine Lines & Wrinkles

**Enlarged** 

**Pores** 



Skin Rejuvenation



Hyperpigmentation



Active Acne & Acne Scars



Dark Circles



Couperose & Sensitive Skin



Irritated & Inflamed Skin



Rosacea

Thinning Hair & Hair Loss



**Excessive**Sweating



Scar Tissue & Stretch Marks

# **INDICATIONS**

JetPeel treatments are ideal for all skin types, improve overall skin health, and can be customized to effectively address a wide variety of skin conditions.

# THE RESULTS ARE VISIBLE

### **DARK CIRCLES & SKIN BRIGHTENING**



Before



After 4 Treatments

### WRINKLE REDUCTION



Before



After 3 Treatments

### **SKIN REJUVENATION**



Before



After 1 Treatment

## FEMALE HAIR RESTORATION



Before



After 10 Treatments

### **ACTIVE ACNE**



**Before** 




After 2 Treatments

### MALE HAIR RESTORATION



Before

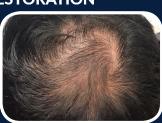

**After 8 Treatments** 



### **ABOUT NYLO AESTHETICS**

NYLO Aesthetics, a new division of NY Laser Outlet, is a leading distributor of aesthetic medical lasers and technology. With an expansive showroom in NYC, as well as offices in Boston and Palm Beach, NYLO sources and distributes innovative, state-of-the-art aesthetic devices and consumables to U.S. medical offices, including dermatologists and plastic surgeons, as well as premier medi-spas, luxury hotel spas, and day spas.

AS SEEN IN

# BRIDES PACTORS VIOLET GREY [ ] [ ]



New York | Boston | Palm Beach

### **NEW YORK CITY SHOWROOM**

- **(\*)** 1.800.887.1966
- 15 W 28th St | Suite 10B | New York, NY 10001
- contactus@nyloaesthetics.com
- www.nyloaesthetics.com